

# Walking Aids

Rollators	714 - 724
Rollator Accessories	725
Tri-Wheel Walkers	726 - 728
Walking Frames	729 - 734
Walking Frame Accessories	735
Buckingham Walking Frame Caddy	736
Stick Seats	737
Walking Sticks	738 - 746
Walking Stick Accessories	747
Crutches	748 - 750

# Four-Wheeled Rollator



## 1 Days Four-Wheeled Rollator

The Four-Wheeled Rollator from Days combines superb quality with fantastic value for money. It features height adjustable handles to ensure the correct position for the user. The brakes provide safe stop and lock into place for security. A padded seat and backrest provide comfort when sitting. The handy basket can carry shopping and personal items. It can hang under the seat or hook onto the front of the frame (allowing the rollator to be folded with the basket in place). The large, swivel wheels ensure the rollator is easy to manoeuvre. The aluminium frame is lightweight but strong enough to withstand everyday wear and tear. It folds easily for transport and storage.

### Specifications

Handle Height	780-900mm (30 to 35½")
Overall Width	605mm (23¾")
Seat Height	535mm (21")
Seat Width	355mm (14")
Seat Depth	315mm (12½")
Wheel Diameter	200mm (8")
Weight	6.8kg (15lb)

Maximum user weight



### Ordering Information

Quartz	091169952
Ruby	091169945

# Fortis Rollator

Sturdy design that adapts to you



## 1 Days Fortis Rollator

The Days Fortis Rollator benefits from a robust design and feels incredibly sturdy when in use. **The seat height can be adjusted independently** so it can be customised to achieve the most comfortable sitting position.

### Quick Release Front Wheels

The 20cm / 8" puncture proof solid PVC wheels "push fit" into the frame and detach quickly and easily using the simple wheel release button.

### Kerb Climbers

Just step down on the kerb climber to tilt the Pulse Rollator back so that the front wheels can easily be lifted onto a kerb.

**Folds in a standing position**, 22cm wide making it easy to store. Working height ready for use. Can also be partially folded when required to fit through tight spaces.

**Folds into a compact size**, for easier storage and transportation

### Specifications

Overall Height	82-97.5cm / 32¼"
Overall Width	61cm / 24"
Overall Depth	61cm / 24"
Seat Height	50-59.5cm / 19¾ - 23½"
Seat Width	39.5cm / 15 ½"
Seat Depth	20cm / 7¾"
Width between Arms	42cm / 16½"
Diameter of Wheels	20cm / 8"
Weight	8.4kg / 18.5lbs
Bag Load Limit	5kg / 11lbs

Maximum user weight **136 kg** **21½ st**


### Ordering Information

Silver	091558345
Blue	091558568



FOLDS WHILE IN A STANDING POSITION

KERB CLIMBERS

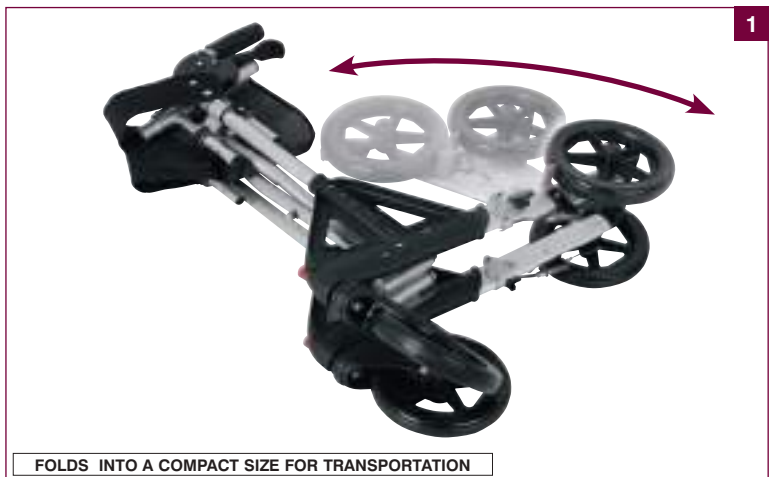

FOLDS INTO A COMPACT SIZE FOR TRANSPORTATION

# Pulse Rollator

Stylish and simple to adjust



## 1 Days Pulse Rollator

The Days Pulse Rollator blends style and function. It is manufactured from elliptical lightweight aluminium tubing and available in two colours – sleek silver and vibrant blue.

### Simple Height Adjustment

The handle height is quick and easy to adjust by lifting the height lock latch so that the handle height can be adjusted into position, then closing to lock into position. The handles also have height setting markings to speed up adjustment.

### Quick Release Front Wheels

The 20cm / 8 inch puncture proof solid PVC wheels "push fit" into the frame and detach easily using the simple wheel release button.

### Kerb Climbers

Just step down on the kerb climber to tilt the Pulse Rollator back so that the front wheels can easily be lifted onto a kerb.

**Folds in a standing position, 25cm wide** making it easy to store and at working height ready for use. Can also be partially folded when required to fit through tight spaces.

**Folds into a compact size**, for easier storage and transportation.

### Specifications

Overall Height	82-95cm / 32¼-37½"
Overall Width	62cm / 24½"
Overall Depth	78cm / 30¾"
Seat Height	56cm / 22"
Seat Width	43cm / 17"
Seat Depth	22cm / 8¾"
Width between Arms	46cm / 18"
Diameter of Wheels	20cm / 8"
Weight	7.9kg / 17.5lbs
Bag Load Limit	5kg / 11lbs

Maximum user weight **136 kg** **21½ st**

### Ordering Information

Silver	091558352
Blue	091558576



# Deluxe Rollator

Sleek, modern and compact



Maximum User Weight  
**1** Colour Available  
**2** Seat Heights Available  
 Silver

## Key Features



**Folds into a standing position, 28cm wide** making it easy to store. Working height ready for use. Can be folded to fit through tight spaces



**Folds into a compact size, 66cm high, 28cm wide, 39cm deep** for easier storage and transport.



**Walking stick holder** supplied as standard.



**Adjustable Height Handles** with 6 settings at 25mm intervals.



**Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.




**Kerb Climbers** just step down on the kerb climber and with a gentle pull back on the handles, the Deluxe Rollator will raise its front wheels to easily manoeuvre over obstacles.



**Built in rest seat** with canvas backrest.



**Comprehensive Spares Available**  
Spares are available for this product, for details and full spares listing please contact Customer Services.

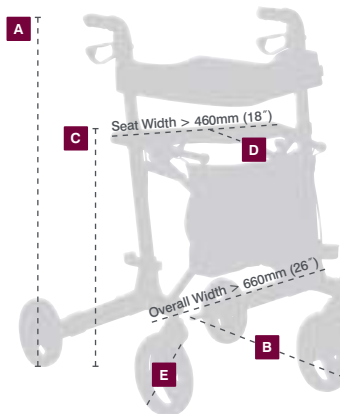

**A Handle Height**  
810 - 930mm / (31¼ - 6½")

**B Overall Depth**  
680mm / (26¾")

**C Seat Height**  
550mm / (21¼")

**D Seat Depth**  
200mm / (7¾")

**E Wheel Diameter**  
195mm front / 165mm rear

**Product Weight**  
7kg

Maximum User Weight

Ordering Information	Seat Height	
Silver	550mm	091554484
	610mm	091439314

# Breeze Indoor Rollator

A smooth journey from room to room



## Days Breeze Indoor Rollator

The Breeze Indoor Rollator is a stylish alternative to a household trolley. Manufactured from painted aluminium tubing with four solid tyre wheels, it incorporates a storage bag and locking brakes. This walker also has height adjustable handles and can be folded for storage or transportation.

### Key Features



**Easy to adjust handle height.**



**Folds down for storage.**



**Simple brake operation** can be applied with just one hand.



**Supplied with a sturdy storage bag and tray.**



**Brakes can be locked on for safety.**

### Specifications

Overall Depth	66cm / 30"
Overall Width	54cm / 21 1/4"
Overall Height	83.5-94.5cm / 32 3/4 - 37 1/4"
	(5 increments)
Tray Depth	29cm / 11 1/2"
Tray Width	39cm / 15 1/2"
Tray Height	68cm / 26 3/4"
Front / Rear Wheel Size	17.75cm / 7"
Weight	5.5kg / 12lbs
Tray Max Load	5kg / 11lbs
Bag Max Load	5kg / 11lbs

Maximum User Weight



### Ordering Information

091558360

# 100 Series

Engineered to stringent standards

The 100 Series aluminium frame ensures a combination of lightweight design and robust build quality with a maximum user weight of 165kg (26st). The range is now available in 6 attractive colours.



**165 kg**  
**26 st**  
 Maximum User Weight

**6**  
 Colours Available

Ruby Red
 Blue
 Quartz
 Racing Green
 Purple
 Russet Orange



## Your Guide to Choosing a Rollator



**It is very important to have the frame at the correct height for use.**

- If the frame is too high, you may find it difficult to straighten your elbows sufficiently and may not take enough body weight through your arms.
- If the frame is too low, it will encourage you to be bent over in a poor posture.
- When being measured for the height of your walking frame wear appropriate and supportive footwear.

Generally, to ensure that the pushing handles are in the best position for weight bearing, the height of the handgrips should be at the level of the wrist bone when the user's elbows are very slightly bent (at an angle of about 15° flexion).

If purchasing a rollator with a seat, the appropriate seat height should be considered. When seated comfortably your feet should be firm and flat on the ground.

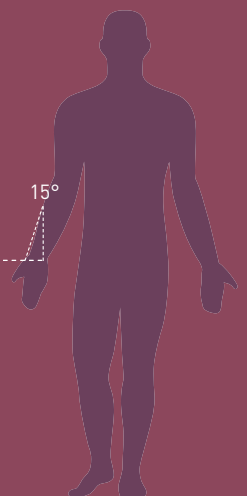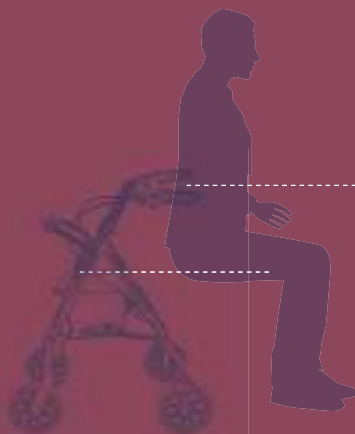

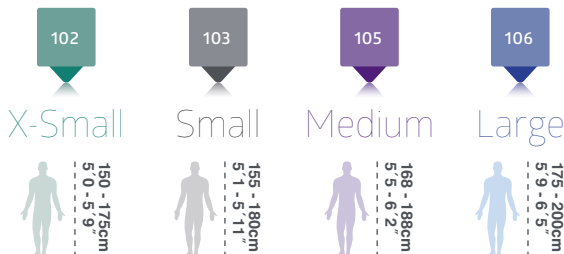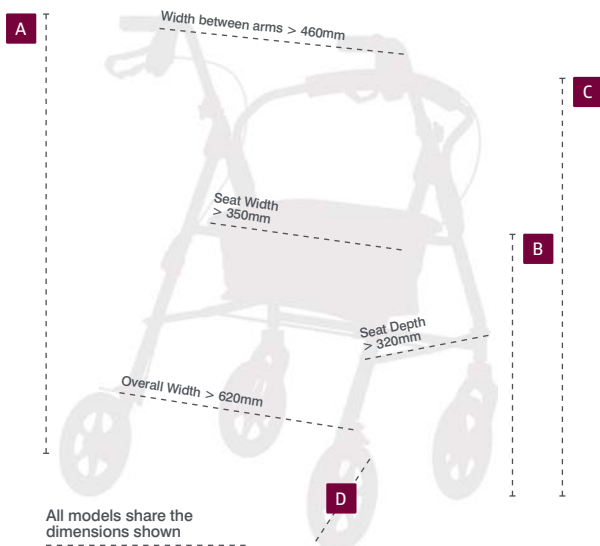

## Key Features



Available in 6 attractive colours.



**Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.



Available in four sizes to accommodate a range of heights and builds.



**Folds Easily for Storage or Transport**  
Simply fold together by pulling the strap on the seat.



**Comprehensive Spares Available**  
Spares are available for this product, for details and full spares listing please contact Customer Services.



**Ergonomic Handgrips & Adjustable Height Handles**  
with 5 settings at 25mm intervals.



## Ordering Information

	102 X-Small	103 Small	105 Medium	106 Large
Ruby Red	091553049	091553106	091356328	091553189
Blue	091553023	091553114	091356336	081561661
Quartz	091553080	091553098	091553163	091553197
Racing Green	091356302	091553148	091553171	091553221
Purple	091553072	091553130	091546563	091553213
Russet Orange	091553056	091553122	091553155	091553205

A	Handle Height 775 - 875mm 30½ - 34½"	Handle Height 780 - 900mm 30¾ - 35½"	Handle Height 840 - 940mm 33 - 37"	Handle Height 875 - 995mm 34½ - 39¼"
B	Seat Height 500mm / 19¼"	Seat Height 535mm / 21"	Seat Height 565mm / 22¼"	Seat Height 620mm / 24¼"
C	Backrest Height 750mm / 29½"	Backrest Height 800mm / 31½"	Backrest Height 800mm / 31½"	Backrest Height 870mm / 34 ¼"
D	Wheel Diameter 150mm / 6"	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"	Wheel Diameter 200mm / 8"
	Product Weight 6kg	Product Weight 7kg	Product Weight 7.2kg	Product Weight 7.3kg

## Accessories Available



### Walking Stick / Crutch Holder

This simple attachment for a walker or wheelchair keeps a walking stick readily available when required. The stick rests in the lower cup section and clips into the top half of the holder to secure it into place. The holder is easily attached to most types of wheelchair or walker. Weight 115g.

**091117779**



### Closure Strap

This useful closure strap keeps your rollator neatly folded together when being stored or transported.

**091558105**



### Reflective Strips

Use this pack of reflective strips to make your rollator or tri-wheeler more visible in the dark. Supplied as a pack of 10.

**091558048**



### Kerb Climber

Navigating kerbs can be awkward with a wheeled walker. This useful add on allows rollators and tri-walkers to be tipped with ease to raise them onto the kerb quickly and easily.

**091558097**



### Bell

This simple bell attaches easily to the handle tubing.

**091558063**



### Bottle Holder

This handy bottle holder makes it easy to stay hydrated when you are out and about. Simply clips onto the frame of your rollator or Tri-Wheeler walker allowing drink bottles to be stored safely and conveniently.

**091558055**

# 2-in-1 Rollator & Transit Chair

Comfort when you need it



**160 kg**  
**25 st**  
Maximum User Weight

**2**  
Colours Available

Silver

Ruby Red

## Key Features



**Available in 2 attractive colours**



**Flip up footrests** provide support during transport.



**Under seat bag** provides useful storage.



**Comfort When You Need It** The seat and backrest are padded for comfort when seated.



**Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.



**Ergonomic Handgrips & Adjustable Height Handles** with 5 settings at 25mm intervals.




**Folds Easily for Storage or Transport** Simply fold together by pulling the strap on the seat.



**Comprehensive Spares Available** Spares are available for this product, for details and full spares listing please contact Customer Services.



**2-in-1** Suitable for use as a rollator or transit chair.

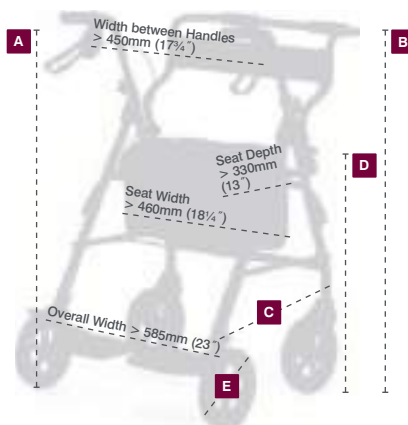

**A Rollator Handle Height**  
800-930mm / 31 3/4"-36 1/2"

**B Pushing Handle Height**  
905mm / 35 1/2"

**C Overall Depth**  
740mm / 29 1/4"

**D Seat Height**  
560mm / 22"

**E Wheel Diameter**  
100mm / 8"

**Product Weight**  
8.8kg

**130 kg** **20 1/2 st** Maximum User Weight

### Ordering Information

Silver	091207299
Ruby Red	091207307

# Quick Fold Rollator

Stylish and lightweight

The Days Quick Fold rollator is a stylish and lightweight rollator that is incredibly easy to fold. The walker folds easily in two directions into an extremely compact size for storage or transportation.



**136 kg** **21 1/2 st**  
 Maximum User Weight

**2**  
 Colours Available


Silver

**Blue**

## Key Features



**Folds in a standing position, 24cm wide** making it easy to store and at working height ready for use. Can also be partially folded when required to fit through tight spaces.

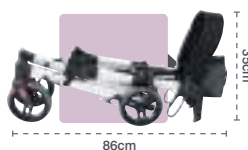

**Folds into a compact size, 86cm high, 24cm wide, 35cm deep** for easier storage and transport.



**Quick Fold Design**  
Folds quickly and easily in two directions using the simple "Quick Fold" lever.



**Integral Reflectors for added visibility**



**Walking Stick Holder** supplied as standard.




**Quick release front wheels**



**Kerb climbers** on the rear legs help to manoeuvre onto kerbs.



**Robust Satchel style bag**

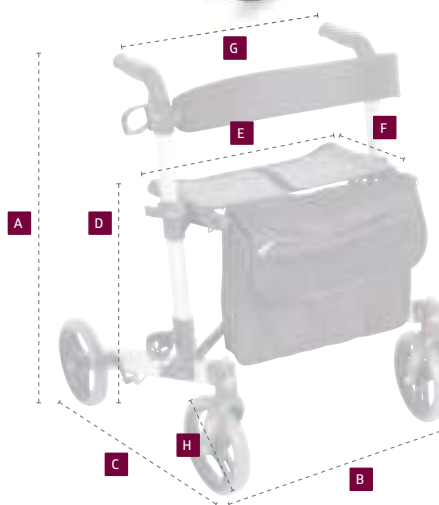

**136 kg** **21 1/2 st** Maximum User Weight

<b>A</b> Handle Height 815-945mm (32 - 37 1/4")	<b>F</b> Seat Depth 225mm (8 3/4")
<b>B</b> Overall Width 640mm (25 1/4")	<b>G</b> Width between Arms 460mm (18")
<b>C</b> Overall Depth 675mm (26 1/2")	<b>H</b> Wheel Diameter 200mm (8")
<b>D</b> Seat Height 560mm (22")	<b>Product Weight</b> 6.42kg
<b>E</b> Seat Width 460mm (18")	<b>Bag Load Limit</b> 5kg

### Ordering Information

Silver	<b>091563980</b>
Blue	<b>091563998</b>

# Heavy Duty Rollators









Our heavy-duty rollators are manufactured using heavy gauge steel tubing making them ideal for larger users.

Built for Strength




<b>Small</b> 091439090 178 kg 28 st Maximum User Weight	<b>Large</b> 081561679 317 kg 50 st Maximum User Weight	<b>2</b> Colours Available	<b>Small</b> 091439090 Blue	<b>Large</b> 081561679 Ruby Red
---------------------------------------------------------------------	---------------------------------------------------------------------	-------------------------------	-----------------------------------	---------------------------------------

## Key Features

-  Available in 2 attractive colours
-  Wider Frames & Rest Seats for added comfort.
-  High User Weights
-  Ergonomic Handgrips & Adjustable Height Handles with 5 settings at 25mm intervals.
-  Loop Lockable Brakes provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.
-  Under Seat Basket provides useful storage.
-  Folds Easily for Storage or Transport simply fold together by pulling the strap on the seat.
-  Comprehensive Spares Available For details and full spares listing please contact Customer Services.

1 Heavy Duty Rollator Small - Blue 091439090	2 Heavy Duty Rollator Large - Ruby Red 081561679
<b>A Handle Height</b> 830-940mm 32¾-37"	<b>Handle Height</b> 885-980mm 34¾-37½"
<b>B Overall Depth</b> 660mm / 26"	<b>Overall Depth</b> 725mm / 28½"
<b>C Overall Width</b> 670mm / 26½"	<b>Overall Width</b> 770mm / 30¼"
<b>D Seat Height</b> 550mm / 21½"	<b>Seat Height</b> 590mm / 23¼"
<b>E Wheel Diameter</b> 200mm / 8"	<b>Wheel Diameter</b> 200mm / 8"
<b>Product Weight</b> 13kg	<b>Product Weight</b> 12.3kg

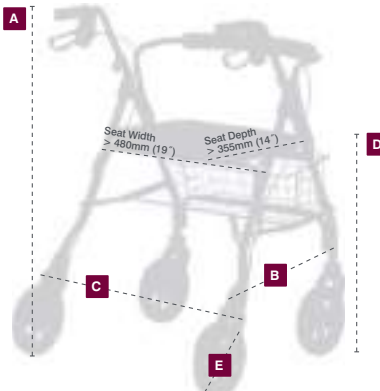

### Ordering Information

- 1 091439090** Blue – Small (178kg)
- 2 081561679** Ruby Red – Large (317kg)

### 3 XXL King Bariatric Rollator

With a weight capacity of 325kg the XXL-Rehab King Rollator is one of the strongest walkers on the market. It fits through doorways and folds easily for transport and storage in cars. It includes a large seat, extra strong tubes and large 8" tyres that provide extra security, stability and comfort. The Rollator has a large support surface with a safety back arch to help reduce the risk of falls.

#### Specifications

- Maximum weight capacity: 325kg
- Seat dimension: 35.5 x 50.8cm
- Width between handles: 61cm
- Total width: 71cm
- Seat height: 50cm
- Adjustable height: 77.5-99cm (2.5cm interval)
- Weight: 15.7kg

091207273

Maximum user weight  



# Essentials 252 Rollator


Ideal for everyday use

Our entry level rollator is ideal for everyday use and provides excellent value for money. It has a lightweight frame, a more compact seat and a lower user weight than the 100 Series, making it ideal for smaller users.



## Key Features

- Ergonomic Handgrips** for optimum comfort.
- Adjustable Height Handles** with 5 individual settings. Each setting is positioned at 25mm intervals.
- Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.
- Comfort When You Need It**  
The seat and backrest are padded for comfort when seated.
- Folds Easily for Storage or Transport**  
Simply fold together by pulling the strap on the seat.
- Comprehensive Spares Available**  
Spares are available for this product, for details and full spares listing please contact Customer Services.

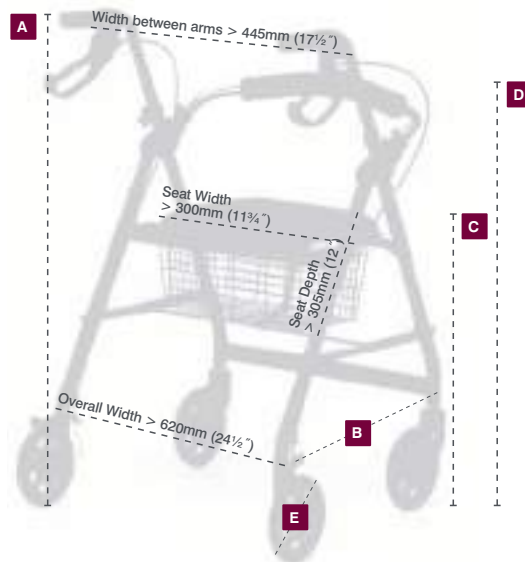

<b>A</b>	<b>Handle Height</b> 795-905mm / 31 1/4-35 1/2"	<b>D</b>	<b>Backrest Height</b> 305mm / 12"
<b>B</b>	<b>Overall Depth</b> 685mm / 27"	<b>E</b>	<b>Wheel Diameter</b> 200mm / 8"
<b>C</b>	<b>Seat Height</b> 530mm / 20 3/4"	<b>Product Weight</b> 6.42kg	

### Ordering Information

091433713 Silver



**1 Days Walking Stick / Crutch Holder**

This simple attachment for a walker or wheelchair keeps a walking stick readily available when required. The stick rests in the lower cup section and clips into the top half of the holder to secure it into place. The holder is easily attached to most types of wheelchair or walker. Weight 115g.

091117779

**2 Days Bottle Holder**

This handy bottle holder makes it easy to stay hydrated when you are out and about. Simply clips onto the frame of your Rollator or Tri-Wheel walker allowing drink bottles to be stored safely and conveniently.

091558055

**3 Days Closure Strap**

This useful closure strap keeps your Rollator neatly folded together when being stored or transported.

091558105

**4 Days Reflective Strips**

Use this pack of reflective strips to make your Rollator or Tri-wheel more visible in the dark. Supplied as a pack of 10.

091558048

**5 Days Kerb Climber**

Navigating kerbs can be awkward with a wheeled walker. This useful add on allows rollators and Tri-Walkers to be tipped with ease to raise them onto the kerb quickly and easily.

091558097

**6 Days Bell**

This simple bell attaches easily to the handle tubing.

091558063



# Steel Tri-Wheel Walkers

Our range of Steel Tri-Wheel walkers are sturdy, robust and economical. Available in 4 attractive colours, plus one model with push down brakes and one model with arthritic attachments.


Sturdy, robust and economical



125 kg / 19½ st Maximum User Weight → 
 1 Year Warranty → 
 4 Colours Available → 
 Ruby Red → 
 Blue → 
 Grey → 
 Chrome

## Key Features

- 4** Available in 4 attractive colours
- Ergonomic Handgrips** for optimum comfort.
- Adjustable Height Handles** with 5 settings at 25mm intervals.
- Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.
- Optional Basket & Tray (091356724)**  
The optional basket and tray provides additional storage and a tray that can be used indoors for use when carrying your lunch from the kitchen.
- Folds Easily for Storage or Transport**  
Using the easy to grip handle that simply pulls the frame together.
- Comprehensive Spares Available**  
Spares are available for this product, for details and full spares listing please contact Customer Services.

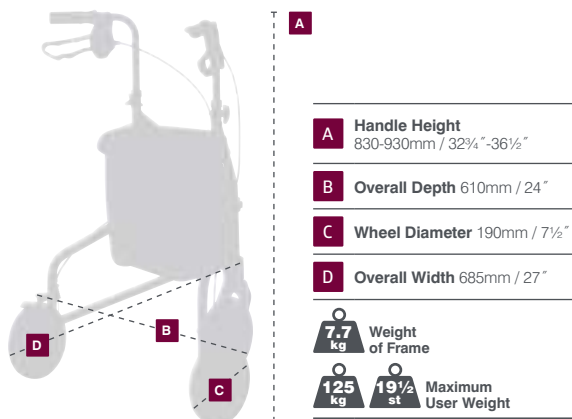

Ordering Information	
Ruby Red	091356765
Blue	091356757
Grey	091439355
Chrome	091325505 <b>NEW</b>
Grey	091117530 With Push Down Brakes
Grey	091117589 With Arthritic Attachments


# 240L Lightweight Tri-Wheel Walkers

The lightweight aluminium frame of these walkers is extremely easy to manoeuvre making them ideal for use indoors or out. The range is now available in 6 attractive colours and supplied with a durable bag as standard.

Lightweight and easy to manoeuvre



Now Available with Bag & Basket

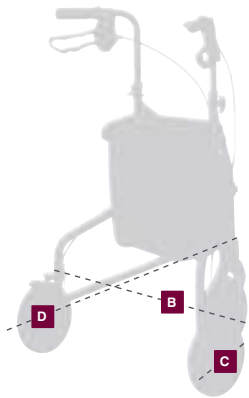

- A Handle Height**  
830-930mm / 32 3/4" - 36 1/2"
  - B Overall Depth** 610mm / 24"
  - C Wheel Diameter** 190mm / 7 1/2"
  - D Overall Width** 685mm / 27"
- 

## Key Features

- Available in 6 attractive colours**
- Now available with or without basket**  
The optional basket and tray provides additional storage and a tray that can be used indoors for use when carrying your lunch from the kitchen.
- Ergonomic Handgrips & Adjustable Height Handles**  
with 5 settings at 25mm intervals.
- Loop Lockable Brakes** provide added security and convenience. The brake tension can also be easily adjusted to suit your requirements.
- Durable Carry Bag** to transport and conceal your belongings.
- Folds Easily for Storage or Transport** using the simple easy to grab pull handle.
- Comprehensive Spares Available** For details and full spares listing please contact Customer Services.

Ordering Information	With Bag Only	With Bag & Basket
Ruby Red	091553015	091557461
Blue	091356732	091557479
Quartz	091553007	091557487
Graphite	091552728	091557495
Purple	091541929	091557503
Russet Orange	091552710	N/A
<b>Accessories</b>		
Basket and Tray	091555242	<b>NEW</b>



**1 Homecraft Three-Wheeled Rollator**

The Three-Wheeled Rollator from Homecraft is extremely durable making it ideal for everyday use. Excellent value for money, it features easy to operate loop brakes, which provide secure stopping and lock into place for security. Height adjustable handles ensure the correct position for the user. It folds easily for transportation and storage and will remain upright when in the folded position. The rollator comes complete with a handy zippered carry bag and large basket with tray. The bag can be left in place when the rollator is folded. The sturdy steel frame is available two attractive colours, ruby and quartz.

**Colour**

**091169978** Quartz

**091169960** Ruby

Maximum user weight

**125 kg** **19 1/2 st**



**RETAIL PACKAGED**  
These products are supplied in retail packaging.



**1 Days Atlas Walking Frame – Standard**



Features a shaped rest pad for increased support.

Chrome plated steel frame is easy to clean.

Brakes on rear castors for additional safety.

**Specifications**

- A** Height: 93-142cm / 36½-56"
- B** Overall width: 63cm / 24¾"
- C** Depth: 76cm / 30"

**091356609**

**Spares**

- D** **091356294** Spare Pad
- E** **091445493** Front Castor
- F** **091351584** Rear Braked Castor

Maximum user weight



**2 Days Wheeled Forearm Walker**



Made from lightweight anodised aluminium.



Rotating handles to reduce strain on wrists.

Moulded armpads for user comfort.

**Specifications**

- A** Height: 96-109cm / 38-43"
- B** Overall width: 74cm / 29"
- C** Depth: 61cm / 24"

**091325372**

**Spares**





- D** **091356237** Arm Pad (single)
- E** **091356120** 25mm PVC Ferrule
- F** **091480268** Extension Leg with Castor

Maximum user weight



Walking Aids



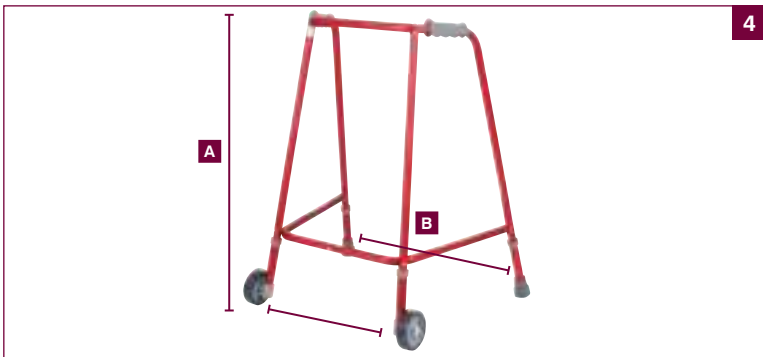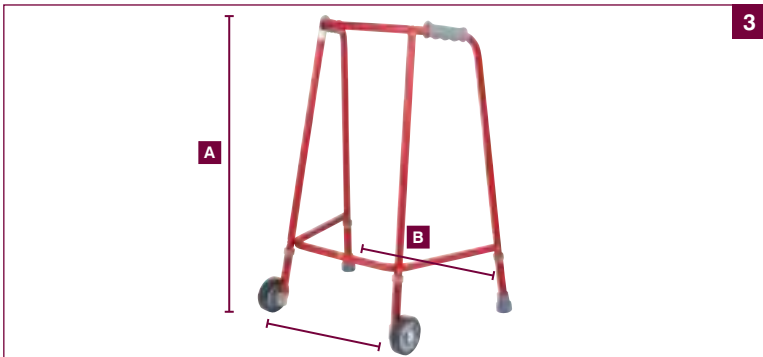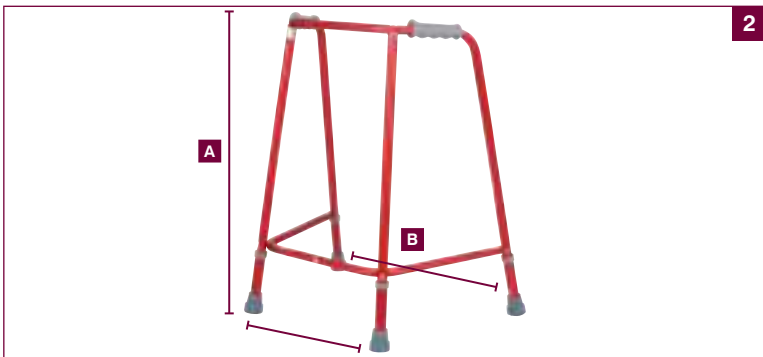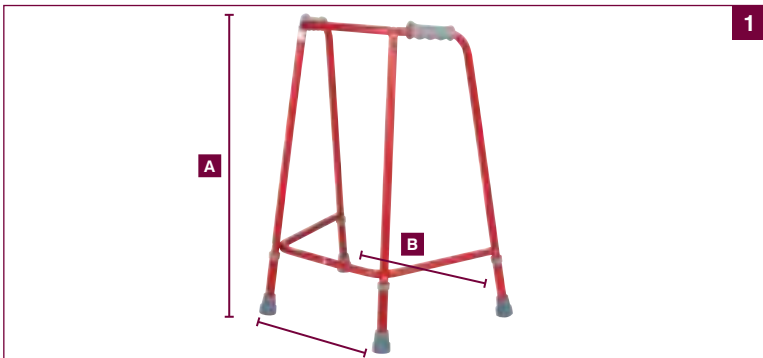










**1** Our most popular walking frames are now available in vibrant metallic red.

These new red Days Walking Frames were developed to meet demand from the NHS market for use with patients with dementia.

Bright colours can highlight important objects, making them easier for people with dementia to find and use, making this range ideal to help increase mobility for patients with dementia and reduce the risk of potential falls.

-  Wheeled and Non-Wheeled versions available
-  Hard anodised finish for durability
-  Comfortable latex free PVC handgrips
-  Low maintenance and easy to wipe clean
-  Lightweight aluminium frame
-  Simple push button height adjustment
-  Non-marking rubber ferrules
- Maximum user weight  

**Days Adjustable Height Walking Frames**

**1 Domestic/Narrow**

- Size
- 091553890** Small
  - 091553866** Medium
  - 091553882** Large

**2 Standard Hospital**

- Size
- 091553775** Small
  - 091553767** Medium
  - 091553759** Large

**3 Domestic/Narrow Wheeled**

- Size
- 091553874** Small
  - 091553841** Medium
  - 091553858** Large

**4 Standard Hospital Wheeled**

- Size
- 091553809** Small
  - 091553791** Medium
  - 091553783** Large

**5 Spares For Above (Not illustrated)**

- 091356120** 25mm PVC ferrule
- 091553981** Wheeled Extension Legs

Specifications						
	Size	Overall Width <b>B</b>	Width at Front	Depth <b>C</b>	Min. Height <b>A</b>	Max Height <b>A</b>
<b>091553890</b>	Small	59cm (23½")	41cm (16")	46cm (18")	70 cm (27½")	78 cm (30¾")
<b>091553866</b>	Medium	60cm (23¾")	42cm (16½")	46cm (18")	80.5 cm (31¾")	88.5 cm (34¾")
<b>091553882</b>	Large	62cm (24½")	42cm (16½")	46cm (18")	87.5 cm (34½")	95.5 cm (37½")
<b>091553775</b>	Small	65cm (25½")	46cm (18")	49cm (19¼")	67cm (26½")	75cm (29½")
<b>091553767</b>	Medium	66cm (26")	47cm (18½")	55cm (21½")	77cm (30¼")	85cm (32½")
<b>091553759</b>	Large	67cm (26½")	51cm (20")	57cm (22½")	87cm (34¼")	95cm (37½")
<b>091553874</b>	Small	59cm (23½")	50cm (19¾")	49cm (19¼")	73cm (28¾")	81cm (31¾")
<b>091553841</b>	Medium	60cm (23¾")	51cm (20")	49cm (19¼")	83.5cm (33")	91.5cm (36")
<b>091553858</b>	Large	62cm (24½")	51cm (20")	49cm (19¼")	90.5cm (35¾")	98.5cm (38¾")
<b>091553809</b>	Small	65cm (25½")	55cm (21½")	52cm (20½")	70cm (27½")	78cm (30¾")
<b>091553791</b>	Medium	66cm (26")	56cm (22")	58cm (22¾")	81cm (31¾")	89cm (35")
<b>091553783</b>	Large	67cm (26½")	51cm (20")	57cm (22½")	87cm (34¼")	95cm (37½")

All Days walking frames are manufactured from lightweight anodised aluminium that is extremely durable and easy to wipe clean. The plastic handgrips have an ergonomic indented shape for greater comfort in use. The non-slip rubber ferrules are non-marking and provide grip and stability.

## 4 Styles, each in up to 3 sizes

A range of widths and heights are available to suit a variety of environments such as hospitals, nursing homes or domestic use where narrower frames are needed due to smaller doorway openings.


 Wheeled and Non-Wheeled versions available	 Hard anodised finish for durability	 Lightweight aluminium frame	 Simple push button height adjustment
 Comfortable latex free PVC handgrips	 Low maintenance and easy to wipe clean	 Non-marking rubber ferrules	

### 1 Days Domestic/Narrow Walking Frame

#### Spares

- D 091356120** 25mm PVC Ferrule Grey Pair
- E 091117357** Wheeled Extension Legs Grey Pair

Maximum user weight

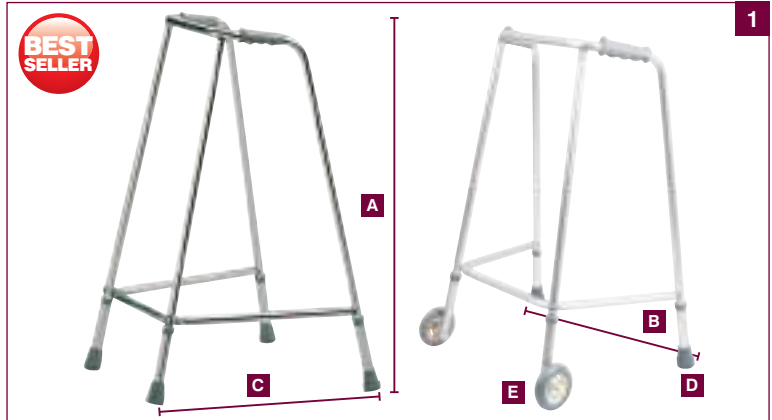

Without Wheels			A Height	B Overall Width	C Depth	Product Weight
Standard	Red					
<b>091325430</b>	<b>091553890</b>	Small	70-78cm / 27½-30¾"	63cm / 24¾"	47cm / 18½"	2.0kg
<b>091356591</b>	<b>091553866</b>	Medium	80.5-88.5cm / 31¾-34¾"	62.5cm / 24½"	46cm / 18"	2.1kg
<b>091325398</b>	<b>091553882</b>	Large	87.5-95.5cm / 34¼-37½"	62cm / 24½"	45cm / 17¾"	2.2kg
With Wheels						
Standard	Red					
<b>091356666</b>	<b>091553874</b>	Small	73-81cm / 28¾-31¾"	63cm / 24¾"	47cm / 18½"	2.1kg
<b>091325497</b>	<b>091553841</b>	Medium	83.5-91.5cm / 33-36"	62.5cm / 24½"	46cm / 18"	2.2kg
<b>091325489</b>	<b>091553858</b>	Large	90.5-98.5cm / 35¾-38¾"	62cm / 24½"	45cm / 17¾"	2.3kg

### 2 Days Standard Hospital Walking Frame

#### Spares

- D 091356120** 25mm PVC Ferrule Grey Pair
- E 091117357** Wheeled Extension Legs Grey Pair

Maximum user weight



Without Wheels			A Height	B Overall Width	C Depth	Product Weight
Standard	Red					
<b>091399849</b>	<b>091553775</b>	Small	67-75cm / 26½ -29½"	65cm / 25½"	53cm / 21"	2.0kg
<b>091356591</b>	<b>091553767</b>	Medium	77-85cm / 30¼ - 32½"	66cm / 26"	54cm / 21¼"	2.1kg
<b>091356575</b>	<b>091553759</b>	Large	87-95cm / 34½ - 37½"	67cm / 26¼"	55cm / 21½"	2.2kg
With Wheels						
Standard	Red					
<b>091438175</b>	<b>091553809</b>	Small	70-78cm / 27½ - 30¾"	65cm / 25½"	53cm / 21"	2.1kg
<b>091438167</b>	<b>091553791</b>	Medium	81-89cm / 31¾ - 35"	66cm / 26"	54cm / 21¼"	2.2kg
<b>091438159</b>	<b>091553783</b>	Large	90-98cm / 35½ - 38½"	67cm / 26¼"	55cm / 21½"	2.3kg



**1 Days Ultra Narrow Frames**

- Wheeled & Non-Wheeled versions available
- Low maintenance and easy to wipe clean
- Comfortable latex free PVC handgrips
- Lightweight aluminium frame
- Simple push button height adjustment
- Non-marking rubber ferrules
- Hard anodised finish for durability

**Spares**

- 091356120** 25mm PVC Ferrule Grey
- 091117357** Wheeled Extension Legs Pair

Maximum user weight **160 kg** **25 st**

Without Wheels		A Height	B Overall Width	C Depth	Product Weight
<b>091356393</b>	Small	72-80cm / 28 1/4-31 1/2"	48.5cm / 19"	44cm / 17 1/4"	2.0kg
<b>091356419</b>	Medium	80-88cm / 31 1/2-34 3/4"	50cm / 19 3/4"	46cm / 18"	2.1kg
<b>091116961</b>	Large	87-95cm / 34 1/4-37 1/2"	52cm / 20 1/2"	47.5cm / 18 3/4"	2.2kg
With Wheels					
<b>091325497</b>	Medium	83-91cm / 32 3/4-35 3/4"	50cm / 19 3/4"	46cm / 18"	2.2kg
<b>091116979</b>	Large	90-98cm / 35 1/2-38 1/2"	52cm / 20 1/2"	47.5cm / 18 3/4"	2.3kg



**2 Days Shallow Frames**

- Wheeled & Non-Wheeled versions available
- Low maintenance and easy to wipe clean
- Comfortable latex free PVC handgrips
- Lightweight aluminium frame
- Simple push button height adjustment
- Non-marking rubber ferrules
- Hard anodised finish for durability

**Spares**

- 091356120** 25mm PVC Ferrule Grey
- 091117357** Wheeled Extension Legs Pair





Maximum user weight **160 kg** **25 st**

Without Wheels		A Height	B Overall Width	C Depth	Product Weight
<b>091116839</b>	Medium	77-85cm / 30 1/4-32 1/2"	62cm / 24 1/2"	37cm / 14 1/2"	2.0kg
<b>091356583</b>	Large	87-95cm / 34 1/4-37 1/2"	62.5cm / 24 1/2"	39cm / 15 1/4"	2.2kg
With Wheels					
<b>091117373</b>	Medium	80-88cm / 31 1/2-34 3/4"	62cm / 24 1/2"	37cm / 14 1/2"	2.2kg
<b>091117381</b>	Large	90-98cm / 35 1/2-38 1/2"	62.5cm / 24 1/2"	39cm / 15 1/4"	2.3kg



**1 Days Adjustable Folding Walking Frame**

Made from lightweight anodised aluminium, this adjustable height walking frame is very easily folded. Non marking, slip-resistant rubber tips for added safety.

-  Anodised aluminium tubing
-  Comfy foam handgrips
-  Non-slip rubber tips
-  Height adjustable legs

**Specifications**

- A** Height: 78-90cm (30-35")
- B** Width at front: 55cm (21")
- C** Depth: 45cm (17")

**091117100**

**Spares**





- D** 091479765 25-22mm Grommet Grey
- E** 091356120 25mm PVC Ferrule Grey

Maximum user weight



**2 Days Reciprocal Folding Walking Frame**

This walker key joints on both sides so that it can "step" forward with each footstep, rather than having to be picked up and placed forwards like a traditional walker.

-  Anodised aluminium tubing
-  Comfy foam handgrips
-  Non-slip rubber tips
-  Height adjustable legs

**Specifications**

- A** Height: 77-90 cm
- B** Width at front: 55cm (21")
- C** Depth: 45cm (17")

**091356682**

**Spares**

- D** 091479765 25-22mm Grommet Grey
- E** 091356120 25mm PVC Ferrule Grey

Maximum user weight





days



## 1 Days Wheeled Forearm Walker

Made from lightweight anodised aluminium this adjustable height wheeled Forearm Walker has moulded rubber armpads for user comfort and rotating handles to reduce strain on wrists. Height adjustment 96 to 109cm (38 to 43"). Overall width 74cm (29")

091325372

Maximum user weight

**160** kg **25** st

## 2 Adjustable Trough Walker

A compact adjustable height aluminium frame with robust troughs lined with PU foam. The handles rotate to the desired position, are height adjustable, and the overarm straps secure the arms in place. External width: 620mm (24 1/2").

	Height	Weight	Style
091117399	830-930mm (32 3/4-36 1/2")	3.2kg	Wheeled
091117407	980-1080mm (38 1/2-42 1/2")	3.4kg	Wheeled

Maximum user weight

**160** kg **25** st

## 3 Folding Bariatric Walking Frame

This walking frame accommodates larger individuals up to 220kg (34 1/2st). The frame is lightweight but solid and offers a high degree of stability. It is height adjustable from 760 to 1010mm (30 to 39 3/4"). Width at base 710mm (28 1/2"). Width between grips 520mm (20 1/2"). Weight 3kg. Folds easily for transport and storage.

091117118

Maximum user weight

**220** kg **34 1/2** st

B

## 4 Heavy Duty Walking Frame

This walking frame offers excellent stability and is designed to accommodate users up to 223kg (35st). Although reinforced, it is made from aluminium ensuring it remains sturdy yet lightweight. It is available in either non-wheeled or wheeled, both height adjustable in 20mm (3/4") increments. Width 700mm (27 1/2"). Depth 420mm (16 1/2").

	Height	Weight	Style
091183516	790-890mm (31-35")	2.6kg	Non-Wheeled
091183524	790-890mm (31-35")	2.8kg	Wheeled
091208156	870-970mm (34 1/4-38")	3.0kg	Non-Wheeled
091208164	870-970mm (34 1/4-38")	3.2kg	Wheeled

Maximum user weight

**223** kg **35** st

B

## 5 Days Hi-Riser Folding Frame

This lightweight folding frame is designed to help the user rise from a seated position. May also be reversed and used as a toilet surround rail. Height adjustable 812 to 914mm (32 to 36"). Width 597mm (23 1/2"). Weight 2.5kg.

091116862

Maximum user weight

**136** kg **21 1/2** st



days



days

**B BARIATRIC**  
These products are designed for bariatric users

## 1 Days Net Bag for Walking Frames

Knotted nylon bag that clips to any walking frame for shopping or daytime requirements. Weight 60g.

091116813



## 2 Days Wheeled Extension Legs

Wheeled extension legs for adjustable height walking frames. Use as a replacement or turn a non-wheeled walking frame into a wheeled one.

091117357

## 3 Walker Skis

These Walker Skis are intended for use on the rear legs of wheeled walkers to help users move smoothly across most surfaces and greatly reduce the friction that rubber tips create.

091559392

## 4 Days Walker Basket

A vinyl coated wire basket, ideal for carrying items, leaving the hands free to move the walker. Attaches easily and safely to the walker frame using the hook and loop straps supplied.

Basket 405×150×185mm.

091117761




days



days



3

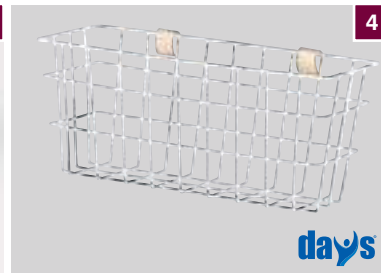

4

days

**RETAIL PACKAGED**  
These products are supplied in retail packaging.

## Days Ferrules



Style	Colour	Pack Type	13mm (1/2")	16mm (5/8")	19mm (3/4")	22mm (7/8")	25mm (1")
5 Standard	Black	Single	-	-	091208842	-	-
		10 Bulk	-	-	091208859	-	-
6 Heavy Duty	Black	Single	-	091168228	091168236	-	-
		10 Bulk	-	091116235	091116250	-	-
7 Shock Absorbing	Black	Single	-	-	091168285	091168293	-
		10 Bulk	-	-	091116474	091116490	-
8 Bell Shaped	Black	Single	091168178	091168186	091168194	091168202	091168210
		10 Bulk	091116078	091116094	091116128	091116144	091116177
9 Heavy Duty Bell Shaped	Grey	Single	-	091168244	091168251	091168269	091168277
		10 Bulk	-	091116367	091116383	091116409	091116425
10 Hard Wearing PVC	Grey	Single	-	091356187	091356245	091356153	091356120
		10 Bulk	-	-	-	091172279	091172287



## 1 Buckingham Walking Frame Caddy

Developed by healthcare professionals to enable users of wheeled walking frames to safely transport a variety of daily living items. In most instances it can replace the need for a trolley. The caddy enables a patient to carry most items safely and more importantly, independently without being reliant upon carers.



▶ One large and one small compartment to allow the user to separate items such as glasses, keys, medication, etc.



▶ Mug holder insert catches any spillages.



▶ Shaped tray that is designed to safely transport a plate of food.



▶ Anti-spill mug cover is also included to prevent any inadvertent spillages when transporting a mug or cup. This simple cover creates a seal and helps to keep liquids inside as well as insulating hot drinks.



▶ Easy to fit to most styles of wheeled walking frame.



▶ Manufactured from a tough, durable plastic it can be cleaned in a dishwasher and has drainage holes for easy drying.

**091117811** Retail Packed  
**091117860** Bulk - Pack of 10

Maximum load weight **2 kg** **4 1/2 lbs**



**SPARES**  
 Spares are available for this product, for details and full spares listing please contact Customer Services.

**RETAIL PACKAGED**  
 These products are supplied in retail packaging.

**1 Height Adjustable Stick Seat**

This stick seat is constructed from aluminium and plastic, making it extremely lightweight to carry and corrosion resistant. It folds easily away, turning into a stick that can be used for support when walking. The most stable seated position is with the handle between the user's legs. Handle height 845 to 950mm (33 1/4 to 37 1/2"). Seat height 455 to 540mm (18 to 21 1/4"). Weight 900g.

**091115658**

Maximum user weight



**2 Days Folding Stick Seat**

This useful seat folds flat and straps together to form a walking stick, offering both support and a place to rest when needed. Manufactured from aluminium, it is both lightweight and corrosion resistant.

**Seat Height**

**Weight**

**091115666** 435mm (17")

1kg

**091115674** 510mm (20")

1.3kg

**091115682** 585mm (23")

1.5kg

**091309905** Spare ferrule

Maximum user weight



**3 Days Folding Tripod Stick**

This attractive seat provides a useful place to rest when out and about. The tripod base ensures it is stable whilst the moulded seat offers comfort and support when needed. Folding away quickly and easily, it can also be used as a walking stick. Seat height 575mm (22 1/2"). Weight 800g.

**091115690**

Maximum user weight



1



2

days



3

days


The extensive range of Days walking sticks provides a variety of styles to suit individuals with varying requirements. Use the guide below to help select the most suitable walking stick in the range.

### Walking Stick Height

A correctly measured stick enables the user to maintain an upright posture with the elbow slightly bent. Body weight is taken through the stick as the user pushes down on it when walking.

Using a walking stick the correct height is essential. When measuring, the handle should rest on the floor against a relaxed arm. The stick should then be adjusted or trimmed to the crease of the wrist.

Where two walking sticks are needed, the length should be slightly longer as they will be used ahead of the body when walking.

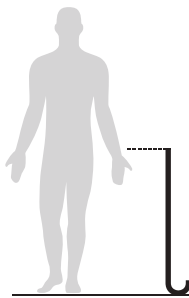

### Which Hand?

If you are using the walking stick to compensate for an injury to your leg or a joint on one side, you should hold the stick in the hand on the opposite side to the one that needs support.

If you are using the stick for balance you can use it in either hand.

### Types of Walking Stick

#### Adjustable

Days adjustable walking sticks can be adjusted to different heights using a simple spring loaded catches and a telescopic shaft.

#### Tripod & Tetrapod

These walking sticks have a wider footprint to provide more stability. Especially suitable for heavier users who require a more stable base. See page 742.

#### Folding Walking Sticks

These lightweight metal sticks have sectioned shafts that enable them to be folded up for storage, for example in a handbag. Strong elastic runs inside the shaft to ensure that in its open position the stick remains stable. See pages 744-746.

#### Material

Our range of walking sticks are available in wood for a traditional look, aluminium and steel.

### Handle Shape

A variety of different shaped handles are available including:

#### Crook Handles



These may be less comfortable to hold than a right-angled handle, but can easily be hooked over the arm when not in use. See page 739 and 741.

#### Right Angled Handles or T-Shaped



These are often more comfortable to use than a crook handle. The addition of a wrist strap may be useful to secure the stick when it is not in use. See pages 739.

#### Swan Necked Shafts



Handles with swan necked shafts are offset above the stick, which allows your weight to be evenly spread centrally over the base of the stick, this may be helpful if you require more stability. See page 740 and 742.

#### Comfy Grip & Contoured Handles



These are anatomically shaped handles, which spread the pressure over a wider area of the palm to improve comfort for permanent users or anyone with painful hands. The comfy grip handles provide the largest surface area and the contoured grip is slightly smaller – making it ideal for smaller hands. See pages 743-744.





**1 Days Adjustable Aluminium Walking Stick**

Days adjustable walking sticks are comfortable to use, can be adjusted to different heights using a simple spring loaded catch and a telescopic shaft.

- 091565969** 84-109 cm Long
- 091565951** 71-96.5 cm Standard

**2 Days Walking Sticks**

Walking Sticks are designed to aid people with restricted movement or mobility by helping them to balance or unload weight from the lower extremities while walking.

- 7500252** Left 740-965mm
- 7500253** Right 740-965mm

Maximum user weight



**3 Days Standard Adjustable Walking Stick**

Moulded Plastic Handle.

**A** Height: 74-99cm / 29-39"

**091325588**

Spares

- B** **091479765** 25-22mm Grommet Grey
- C** **091356153** 22mm PVC Ferrule Grey

Maximum user weight



**4 Days Heavy Duty Crook Handle Walking Stick**

Traditional Crook Handle design allows the stick to be hooked over the arm when not in use.




**A** Height: 79-101.5cm / 31-40"

**091356880**

Spares

- B** **091356245** 18mm PVC Ferrule Grey

Maximum user weight

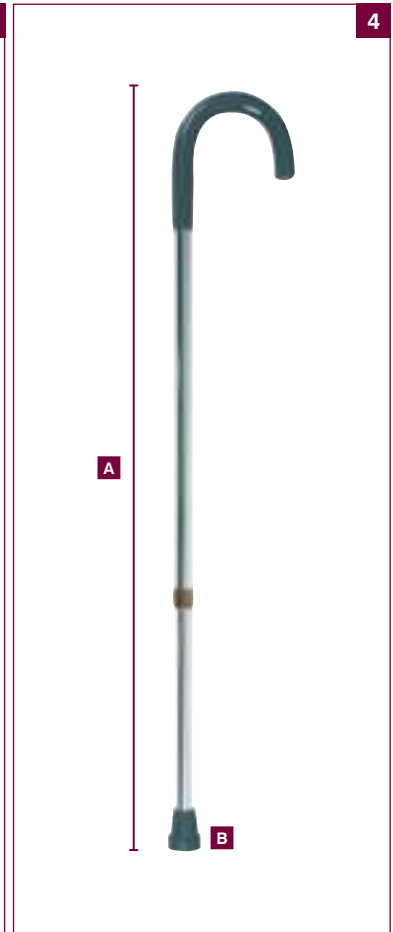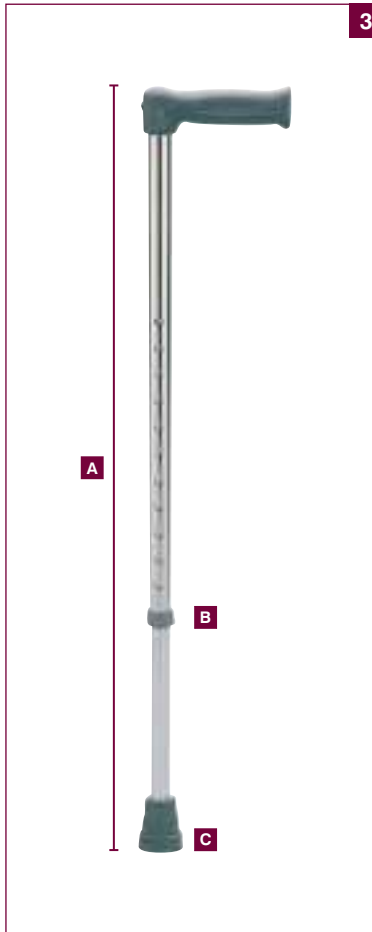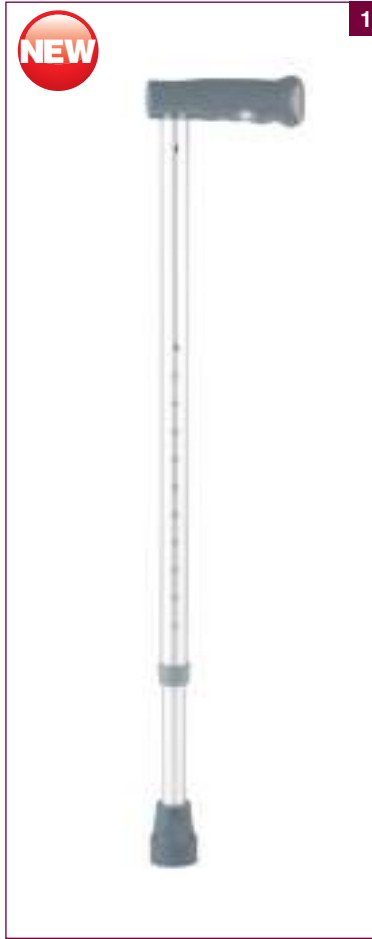



**1 Days Ergonomic Handle Walking Stick**

- Ergonomic Handle spreads the user's weight over a wider area of the hand.
- Shaped for either left handed or right handed users.

**A** Height: 73-96cm / 28½-37½"

**091325513** Left

**091325521** Right

Spares

**B** **091356245** 18mm PVC Ferrule Grey

Maximum user weight



**2 Days Swan Neck Walking Stick**

- Swan neck handle allows the users weight to be transferred vertically downwards.
- Ideal for providing more stability and for user's who apply a lot of weight on their stick.

**A** Height: 74-99cm / 29-39"

**091356872**

Spares

**B** **091479765** 25-22mm Grommet Grey

**C** **091356153** 22mm PVC Ferrule Grey

Maximum user weight



**3 Days Folding Steel Walking Stick**

- Durable powder coated steel shaft.
- Supplied in a polythene wallet for storage.



Folds easily for storage or transport



Simple push button height adjustment



T-Bar handle



Non-marking slip resistant ferrule



2 heights available

**A** Height

**091356914** 85-95cm / 33-37"

**091356922** 74-83.5cm / 29-33"

Maximum user weight



## 1 Bariatric Walking Stick

A height adjustable walking stick designed especially for bariatric users. The curved neck places the user's weight directly over the shaft for stability and safety. Height adjustable from 710-930mm (28-36½"). Weight 0.9kg.

091114982

Maximum user weight



**B**

## 2 Chestnut Wood Walking Stick

This stylish traditional walking stick is made from chestnut and has a traditional crook handle with a rubber ferrule. Three lengths are available.

Height

091540954 91.5cm / 36"

091540962 96.5cm / 38"

091540970 101.5cm / 40"

Maximum user weight



## 3 Crook Handle Walking Stick

These high quality sticks are made from a smooth, dense wood and are available in a choice of either natural beech or black finishes. They are finished in a gloss varnish, making them both durable and attractive. Supplied as a 914mm (36") stick, they should be sawn to the length required. Ferrule size 19mm (¾"). Weight 360g.

091115302 Natural Beech

091115310 Black

Maximum user weight



**B** **BARIATRIC**  
These products are designed for bariatric users


**i** **INFORMATION**  
For information about selecting the correct length of walking stick, please see page 738.

Our range of Quadruped and Tripod Sticks are designed to provide the user with maximum stability and are ideal for anyone who is more unstable on their feet.



## 1 Quadruped Walking Stick with Wide Base

• Styled as a traditional walking stick with a large quadruped base.

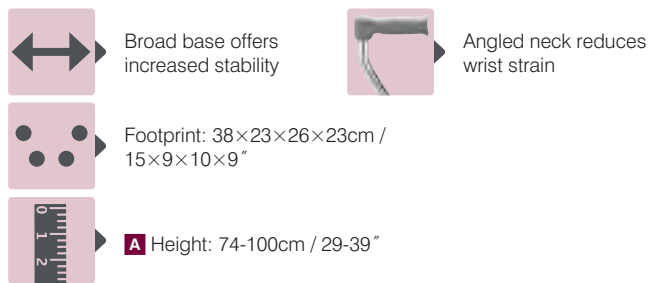

091356971

Spares

**B** 091479765 25-22mm Grommet  
**C** 091356187 16mm PVC Ferrule


Grey

Maximum user weight



## 3 Tripod Walking Stick with Large Base

• Styled as a traditional walking stick with a large tripod base.

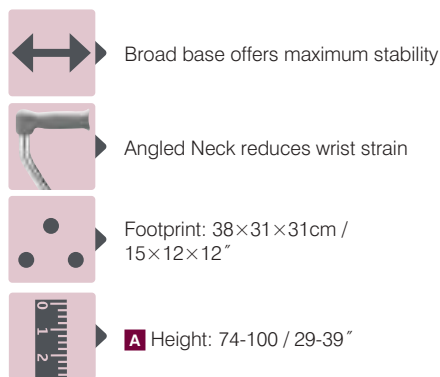

091325539

Spares


**B** 091479773 22-19mm Grommet  
**C** 091356187 16mm PVC Ferrule  
**D** 091443365 Pin Clip

Maximum user weight



## 2 Quadruped Walking Stick with Small Base

• Styled as a traditional walking stick with a small quadruped base.

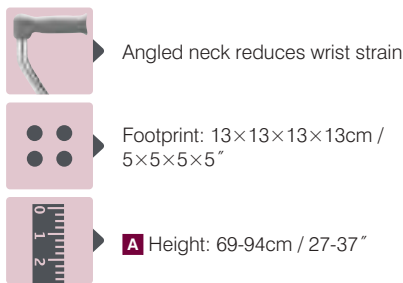

091115344

Spares


**B** 091479773 22-19mm Grommet  
**C** 091356187 16mm PVC Ferrule

Maximum user weight



## 4 Tripod Walking Stick with Small Base

• Styled as a traditional walking stick with a small tripod base.

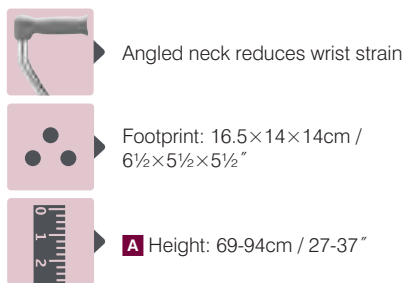

091356930

Spares

**B** 091356187 22-19mm Grommet  
**C** 091356187 16mm PVC Ferrule

Maximum user weight





**1 Homecraft Patterned Walking Sticks**

Available in 7 attractive patterns, these height adjustable walking sticks are of the highest quality. Each stick has received a hard anodised treatment that prevents damage to the attractive surface. The wooden handle is shaped to fit comfortably in either hand. Height adjustable between 790-1020mm (31-40"). Ferrule size 19mm (3/4"). Weight 340g.

- 091114727** Black
- 091175934** Wild Rose
- 091114743** Grey Marble
- 091310010** Etched Black
- 091175942** Woodland Flower
- 091114693** Walnut
- 091175975** Paisley

Maximum user weight



Walking Aids

1



BLACK

WILD ROSE

GREY MARBLE

1




ETCHED BLACK

WOODLAND FLOWERS

WALNUT

PAISLEY

**RETAIL PACKAGED**  
These products are supplied in retail packaging.

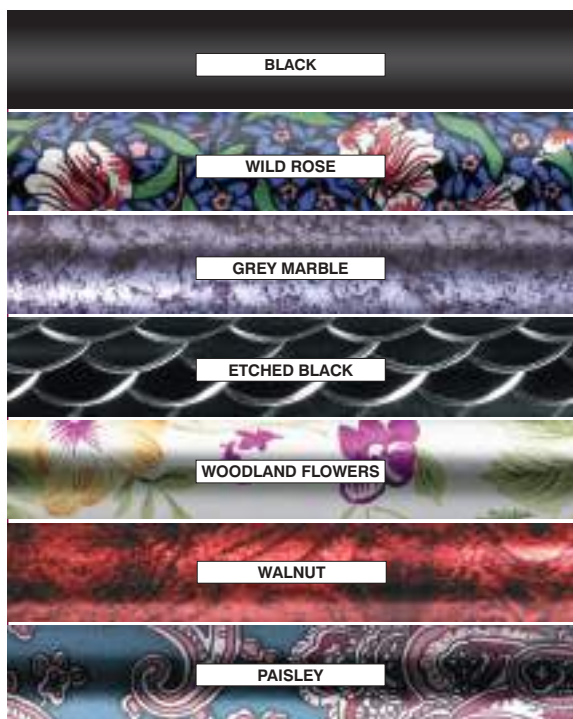

BLACK

WILD ROSE

GREY MARBLE

ETCHED BLACK

WOODLAND FLOWERS


WALNUT

PAISLEY



**1 Homecraft Folding Patterned Walking Stick**

Available in 7 patterns, these folding height adjustable walking sticks have received a hard anodised treatment that prevents damage to the attractive surface. The wooden handle is shaped to fit comfortably in either hand. Made from aluminium, making them extremely lightweight, they are available in three height adjustments. Ferrule size 19mm (3/4"). Weight 360g.

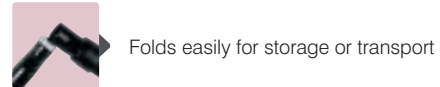

Folds easily for storage or transport



Simple push button height adjustment



Wooden T-Bar handle



Non-marking slip resistant ferrule



3 heights available

Height	Height	Height
735-825mm (29-33")	775-875mm (31-35")	825-925mm (33-37")



091172592	091114925	091114933	Black
091172774	091172717	091172725	Wild Rose
091204692	091204700	091204734	Etched Black
091172659	091114966	091114974	Grey Marble
091172766	091172691	091172709	Woodland Flowers
091172576	091114867	091114875	Walnut
091172758	091172733	091172741	Paisley

Maximum user weight



**RETAIL PACKAGED**  
These products are supplied in retail packaging.

**INFORMATION**  
For information about selecting the correct length of walking stick, please see page 738.

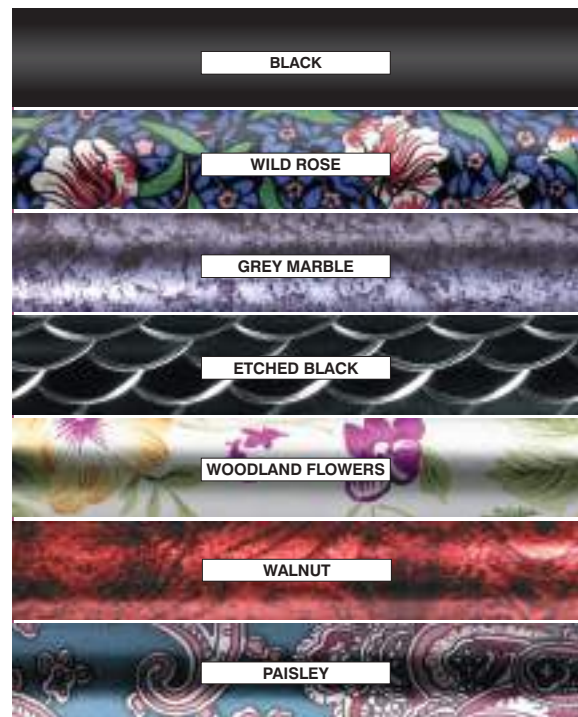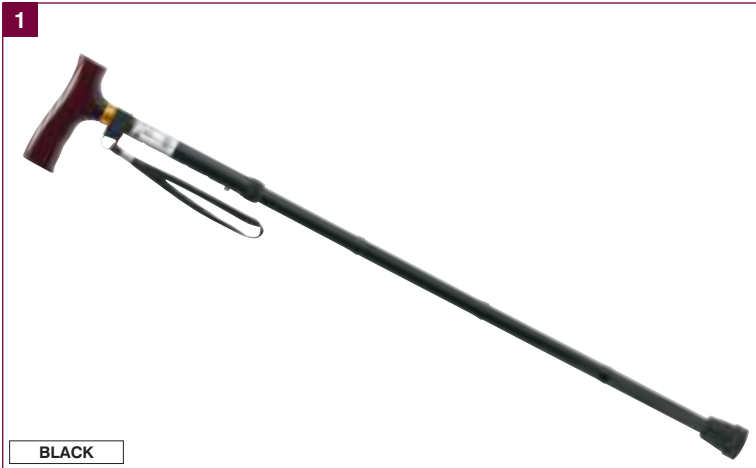



**1 Homecraft Comfy Grip Sticks**

Lightweight, yet sturdy anodised aluminium walking sticks that feature an ergonomically designed handle allowing the pressure to be spread evenly across the palm. Ideal for those who have difficulty in grasping a conventionally shaped handle, the comfortable design can also be beneficial to those with painful hands due to arthritis, or similar problems. Supplied as either left or right handed, standard or folding. Ferrule size 19mm.

**Standard**

Height adjustable between 738-955mm (29-38").

Weight 382g.

Left Handed	Right Handed	Colour
091115138	091115138	Black
091205608	091205616	Etched Black
091205715	091205723	Woodland Flowers

Maximum user weight



**Folding**

Available in short, 775-875mm (31-35") and long 825-925mm (33-37"). Weight 340g.

Left Handed	Right Handed	Length	Colour
091114784	091114792	Short	Black
091114800	091114818	Long	Black
091207968	091207976	Short	Woodland Flowers

Maximum user weight



Walking Aids




BLACK

WOODLAND FLOWERS

LEFT HAND

RIGHT HAND

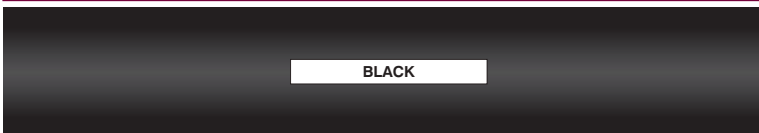

BLACK



WOODLAND FLOWERS

**RETAIL PACKAGED**  
These products are supplied in retail packaging.

**INFORMATION**  
For information about selecting the correct length of walking stick, please see page 738.



### 1 Homecraft Contoured Grip Sticks

The ergonomic handle of these sticks is designed for users with painful or arthritic hands. They spread the weight across the whole palm making it more comfortable than a standard walking stick. It is especially useful for smaller hands, where the Comfy Grip Stick may be too large. The handle is also positioned more toward the centre of the cane, which helps reduce the stress on the wrist. These lightweight aluminium sticks are available in left and right handed options and as standard or folding sticks. Ferrule size 19mm.

#### Standard

Height adjustable between 738-955mm (29-38"). Weight 380g.

Left Handed	Right Handed	Colour
091115260	091115278	Black
091205566	091205574	Woodland Flowers

Maximum user weight



#### Folding

Available in short, 775-875mm (31-35") and long 825-925mm (33-37"). Weight 330g.

Left Handed	Right Handed	Length	Colour
091114826	091114834	Short	Black
091114842	091114859	Long	Black

Maximum user weight



### 2 Homecraft Premium Folding Coloured Walking Sticks

A beautiful addition to the folding coloured walking stick range, available in 3 delightful colours. They feature a smooth Derby handle that is comfortable to use in either hand and a handy strap as standard. The sturdy, aluminium shafts have a transparent coating that enhances the design. Available in two height adjustments. Ferrule size 19mm (3/4"). Weight 366g.

Height	Height	
775-875mm (31-35")	825-925mm (33-37")	
091209097	091209105	Black
091209113	091209121	Ivy
091209170		Flora

Maximum user weight



**RETAIL PACKAGED**  
These products are supplied in retail packaging.

**INFORMATION**  
For information about selecting the correct length of walking stick, please see page 738.

## 1 Homecraft Walking Stick Strap

This nylon wrist strap offers exceptional value for money and is suitable for use with all sticks. The elasticated hoop allows easy and secure fitting. Weight 5g.

091116607



## 2 Walking Stick Grip

A value for money clip that grasps the stick, allowing it to be balanced on the edge of a table or worktop. May also be used underneath the table or worktop to wedge the stick upright. Fits most styles of walking stick. Weight 20g.

091116615

## 3 Walking Stick Clip

This double clip device can be used for walking sticks, reachers and many other applications. Fitted onto a wheelchair or walker using a rubber ring system. The plastisol coated clip securely grips walking sticks and reachers.

091116342

## 4 Walking Stick Holder

This simple attachment for a walker or wheelchair keeps a walking stick readily available when required. The stick rests in the lower cup section and clips into the top half of the holder to secure it into place. The holder is easily attached to most types of wheelchair or walker. Weight 115g.

091117779



**RETAIL PACKAGED**  
These products are supplied in retail packaging.

All Days crutches are made from lightweight aluminium with a tough anodised finish making them incredibly durable. They are designed for minimum maintenance and just require an occasional wipe-over with a soft cloth and can be used with disinfectant solution if required.

### Three Handle Options



#### Standard Handle

The standard plastic hand grip is easily wiped clean to reduce the risk of cross infection.



#### Ergonomic Grip


The Days Ergonomic handle has been designed and developed in conjunction with physiotherapists and healthcare professionals. The unique handle distributes pressure over a greater area of the user's hand improving comfort and safety.



#### Comfy Grip


The Comfy Grip handle is made from a soft touch plastic to provide a more comfortable and cushioned grip – ideal for longer term use.

### 3 Levels of Adjustment

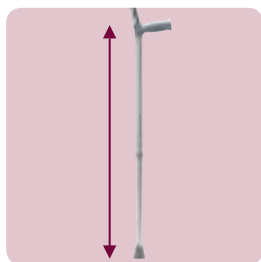

#### Single Adjustable Crutches

Offer height adjustment for the handle height but the distance between the handle and arm cuff is fixed.

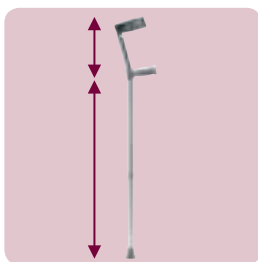

#### Double Adjustable Crutches

Allows you to adjust both the height to handle and the distance between the handle and the arm cuff.



#### Days Permanent Crutches

These user crutches are not adjustable and need to be cut to size to suit the individual user – ideal for long term use.

### Standard Features



#### Closed Fixed Armband

The closed fixed armband enables the user to retain the crutch on their arm even if they release their grip on the handle. This prevents the crutch from dropping to the floor accidentally.



#### Non-Marking Ferrules

The non-marking ferrules are slip-resistant, for added safety. The 22mm ferrules are fitted to the adult crutches and 16mm ferrules are fitted to the children's crutches.



#### Push Pin Adjustment

Adjustment is easy using the simple push pin operation.

**1 Days Standard Grip Crutches**

		<b>A Handle to Armband</b>	<b>B Handle to Ground</b>
Single Adjustable			
<b>091275965</b>	Standard	23cm / 9"	66-94cm / 26-37"
<b>091356468</b>	Extra Long	23cm / 9"	83-110cm / 32½-43½"
Double Adjustable			
<b>091325323</b>	Children's	17-22cm / 6¾-8¾"	56-80cm / 21½-31½"
<b>091115799</b>	Standard	20-27.5cm / 8-10½"	66-94cm / 26-37"
<b>091356443</b>	Extra Long	20-27.5cm / 8-10½"	83-110cm / 32½-43½"

Spares

- C 091479765** 25-22mm Grommet Grey
- D 091356153** 22mm PVC Ferrule Grey
- 091356187** 16mm PVC Ferrule (Childrens Crutches) Grey

**2 Days Ergonomic Grip Crutches**

		<b>A Handle to Armband</b>	<b>B Handle to Ground</b>
Single Adjustable			
<b>091269042</b>	Standard	23cm / 9"	69-94cm / 27-37"
Double Adjustable			
<b>091356484</b>	Standard	20-27.5cm / 8-10½"	69-94cm / 27-37"
<b>091356492</b>	Extra Long	20-27.5cm / 8-10½"	83-110cm / 32½-43½"
Permanent Use / Cut to Size			
<b>091356385</b>		23.5-31cm / 9¼-12¼"	103cm / 40½"

Spares

- C 091479765** 25-22mm Grommet Grey
- D 091356153** 22mm PVC Ferrule Grey

**3 Days Comfy Crutches**

		<b>A Handle to Armband</b>	<b>B Handle to Ground</b>
Single Adjustable			
<b>091439199</b>	Standard	23cm / 9"	69-94cm / 27-37"
<b>091531540</b>	X-Long	23cm / 9"	83-110cm / 32½-43½"
Double Adjustable			
<b>091439207</b>	Standard	66-94cm / 26-37"	20-27.5cm / 8-10½"
<b>091531557</b>	X-Long	83-110cm / 32½-43½"	20-27.5cm / 8-10½"

Spares

- C 091479765** 25-22mm Grommet Grey
- D 091356153** 22mm PVC Ferrule Grey



**£ PRICES**  
For up-to-date prices please see our Website, alternatively contact your Sales Representative or Customer Services.



## 1 Soft Grip Comfort Handle Crutches

These crutches feature a soft anatomically shaped handle for comfort and are available in four modern colours. They are height adjustable between both the arm cuff and handle and the handle and ferrule. Supplied in pairs. Height to handle 730 to 960mm (28<sup>3</sup>/<sub>4</sub> to 37<sup>1</sup>/<sub>4</sub>" ). Handle to cuff 240 to 290mm (9<sup>1</sup>/<sub>2</sub> to 11<sup>1</sup>/<sub>2</sub>" ). Ferrule diameter 18<sup>1</sup>/<sub>2</sub>mm (3/4").

- 091159060** Red
- 091159094** Blue
- 091159102** Turquoise
- 091159110** Grey

Maximum user weight



## 2 Days Arthritic Crutches

Made from lightweight anodised aluminium with adjustable height handles, these arthritic crutches feature an arm trough for user comfort. The rotating handles reduce strain on wrists and the non-marking, slip resistant rubber tips add user safety. Adjustable from 94cm to 119cm (37 to 47"). Supplied as a pair.

**091437888**

Maximum user weight



## 3 Bariatric Crutch

This heavy-duty crutch is designed to for larger individuals weighing up to 325kg (51st). It is made from 28mm lightweight tubing and has a 25mm rubber ferrule to provide a large ground contact area. Height adjustable from 690 to 1100mm (27<sup>1</sup>/<sub>4</sub> to 43<sup>1</sup>/<sub>4</sub>" ). Distance between handle and cuff 240mm (9<sup>1</sup>/<sub>2</sub>" ). Inside curve of cuff is 150mm (6"). Supplied as a single crutch. Weight 1.5kg. Available with a standard handle or ergonomic right or left handed grip.

- 091208909** Standard
- 091525989** Ergonomic Grip Left
- 091525997** Ergonomic Grip Right

Maximum user weight



**B**

## 4 Padded Handle Cover

These useful Neoprene guards help cushion the hands and protect against calluses. Simply place over the crutch handle and zip up. Sold as single units.

**091116334**



**B BARIATRIC**  
These products are designed for bariatric users